medi USA, LP  
6481 Franz Warner Pkwy.  
Whitsett, NC 27377  
USA  
www.mediusa.com

EC REP medi GmbH & Co. KG  
Medicusstraße 1  
95448 Bayreuth, Germany  
www.medi.de

medi Australia Pty Ltd  
83 Fennell Street  
North Parramatta NSW 2151  
Australia  
T +61 2 9890 8696  
F +61 2 9890 8439  
sales@mediaustralia.com.au  
www.mediaustralia.com.au

medi Danmark ApS  
Vejlegardsvej 59  
2665 Vallensbaek Strand  
Denmark  
T +45 70 25 56 10  
F +45 70 25 56 20  
kundeservice@sw.dk  
www.medit danmark.dk

medi Austria GmbH  
Adamgasse 16/7  
6020 Innsbruck  
Austria  
T +43-512 57 95 15  
F +43-512 57 95 15 45  
vertrieb@medi-austria.at  
www.medi-austria.at

medi Bayreuth España SL  
C/Canigó 2 – 6 bajos  
Hospitalet de Llobregat  
08901 Barcelona  
Spain  
T +34 93 260 04 00  
F +34 93 260 23 14  
medi@mediespana.com  
www.mediespana.com

medi Belgium bvba  
Posthoornstraat 13/1  
3582 Koersel  
Belgium  
T: + 32-11 24 25 60  
F: +32-11 24 25 64  
info@medibelgium.be  
www.medibelgium.be

medi France  
Z.I. Charles de Gaulle  
25, rue Henri Farman  
93297 Tremblay en France Cedex  
France  
T +33 1 48 61 76 10  
F +33 1 49 63 33 05  
infos@medi-france.com  
www.medi-france.com

medi Brasil  
Rua Itapeva 538 cj. 42  
Bela Vista, São Paulo - SP  
CEP: 01332-000  
Brasil  
T +55 (11) 3500 8000  
sac@medibrasil.com  
www.medibrasil.com

4 049772 365708

E012043/09.2017

medi

circaid compression anklet

CIRCAID

circaid®  
compression anklet  
elastic compression system

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## circaid® compression anklet

### Purpose

The garment is a single patient, reusable, non-invasive, non-sterile, elastic compression anklet designed to provide therapeutic compression to the foot and ankle region for patients with venous and lymphatic disorders and is always to be worn in combination with lower leg compression garments and if necessary a foot garment.

### Important to note:

Independently of medical diagnosis and prescribed compression treatment, circaid compression anklet is to be worn in combination with foot, lower leg and if necessary upper leg compression garments.

The circaid compression anklet is not defined as "medical

compression stockings" according to RAL-GZ 387:2000-09 Medizinische Kompressionsstrümpfe.

Detailed information on the indications can be found in the respective instruction manuals for the main products circaid juxtacures, circaid juxtalite or circaid juxtaft."

### Storage instructions

Store in a dry place

Keep out of sunlight

The shelf-life date is printed on the carton label with the symbol of a "sand glass".

**This garment is not designed to be cut or modified in any way.**

### Lifespan

Due to material wear-and-tear and the medical efficacy can only be guaranteed for a defined lifespan. This assumes correct

medi Hungary Kft.  
Bokor u. 21.  
1037 Budapest  
Hungary  
T +36 1371 0090  
F +36 1371 0091  
info@medi.hu  
www.medi.hu

medi Italia s.r.l.  
Via Giuseppe Ghedini, 2  
40069 Zola Predosa (BO)  
Italy  
T: +39-051 6 13 24 84  
F: +39-051 6 13 29 56  
info@medi-italia.it  
www.medi-italia.it

medi Nederland BV  
Heusing 5  
4817 ZB Breda  
The Netherlands  
T +31 76 57 22 555  
F +31 76 57 22 565  
info@medi.nl  
www.medi.nl

medi Norway AS  
P.B. 238  
4291 Kopervik  
Norway  
T: +47 52 84 45 00  
F: 47 52 84 45 09  
post@medinorway.no  
www.medinorway.no

medi Orient M.E.A. ZCC  
TAURUS GROUP  
Suite No. M01 Wasel Al Mamzar Bldg.  
Al Wuhida Road, Al Mamzar  
P.O.BOX NO. 91464  
DUBAI, UAE  
United Arab Emirates  
T: +971-4 2557344  
F: +971- 4 2557399  
md@taurusgroup.net

medi Polska Sp. z o.o.  
Zygmunta Starego 26  
44-100 Gliwice  
Poland  
T +48-32 230 60 21  
F +48-32 202 87 56  
info@medi-polska.pl  
www.medi-polska.pl

Step 1

Step 2

Step 3

handling of the garment (e.g. proper care, donning, and doffing).

circaid compression anklet:  
<3 month - depends on the wear and tear

### Disposal

Please dispose of the garment in household waste. No special criteria for disposal apply for medical compression garments.

### Material composition

circaid compression anklet  
88% Nylon  
12% Elastane

This garment does not contain latex.

### Washing instructions

The garment can be machine washed using a gentle cycle and dried on a low heat setting. To extend the life of the garment, hand wash and drip dry.

- 
- Machine wash warm
  - Do not bleach
  - Tumble dry, low heat
  - Do not iron
  - Do not dry clean
  - Do not wring
- 

### Wearing recommendation

If experiencing any pain, immediately remove the garment and consult your physician. The garment should never hurt. It should feel firm but comfortable. The circaid compression anklet may be worn day and night, but removal is recommended if experiencing discomfort when lying down.

### **Donning**

If using the circaid compression ankle make sure that you have already applied a circaid leg garment and a circaid undersleeve. To avoid uncomfortable wearing due to wrinkles in the undersleeve ensure that the bottom part of the circaid undersleeve is folded over the bottom part of the circaid leg garment.

**Step 1:** For an easier application, slide your hand into the circaid compression ankle and turn it inside out just up to the heel.

**Step 2:** Slide the circaid compression ankle onto the foot and ensure that it is pulled high enough that it overlaps with the circaid leg garment.

### **Removal**

Slide the circaid compression ankle off the foot.

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